



Mask/Device Compatibility List

This document specifies the masks compatible with ResMed's positive airway pressure devices and the mask setting, if one is required. For full details regarding the correct use of these devices, please refer to the specific User Guides.

Note: Not all products are available in all regions.

Full Face Masks

Device	AcuCare™ F1-4	AirTouch™ F20 / AirTouch™ F20 for Her	AirFit™ F20 / AirFit™ F20 for Her	AirFit™ F10 / AirFit™ F10 for Her	Mirage Quattro™	Quattro™ FX / Quattro™ FX for Her	Quattro™ Air / Quattro™ Air for Her	Mirage Liberty™	Ultra Mirage™	Hospital Full Face
AirMini™	X	✓	✓	X	X	X	X	X	X	X
Air 10 Series	N/V	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
S9 AutoSet™ / S9 AutoSet™ for Her / S9 Elite™	N/V	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
S9 Escape™ / S9 Escape™ Auto	N/V	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
C-Series Tango™	N/V	N/V	N/V	No setting required. Consult your clinician before changing to a different mask type.						
S7 Elite™	N/V	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 AutoSet Vantage™	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 Compact™	N/V	N/V	N/V	No setting required. Consult your clinician before changing to a different mask type.						
S8 Elite™ / S8 Elite™ II	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 Escape™	N/V	N/V	N/V	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.						
S8 Escape™ II	N/V	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.								
S8 AutoSet™ II	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ III	N/V	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ III ST-A	N/V	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP Adapt SV™	N/V	N/V	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE ¹	FULL FACE	FULL FACE
VPAP™ Adapt (S9)	N/V	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Adapt (S9 ASV)	N/V	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Auto 25	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ Auto (S9)	N/V	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Malibu	N/V	N/V	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE
VPAP™ S / VPAP™ ST	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ S (S9) / VPAP™ ST (S9)	N/V	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ ST-A (S9 iVAPS)	N/V	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ COPD (S9)	N/V	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
S9 VPAP™ Tx	N/V	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face

Device	AcuCare™ F1-4	AirTouch™ F20 / AirTouch™ F20 for Her	AirFit™ F20 / AirFit™ F20 for Her	AirFit™ F10 / AirFit™ F10 for Her	Mirage Quattro™	Quattro™ FX / Quattro™ FX for Her	Quattro™ Air / Quattro™ Air for Her	Mirage Liberty™	Ultra Mirage™	Hospital Full Face
VPAP™ Tx	N/V	N/V	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE
Lumis™ Tx	N/V	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
Stellar™	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE
VS Ultra™ / VS Integra™	✓	N/V	N/V	✓	✓	N/V	✓	✓	✓	✓
VS III™	N/V	N/V	N/V	✓	✓	N/V	✓	✓	✓	✓
Astral™	FULL FACE	FULL FACE	FULL FACE	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE

¹ SmartStop may not operate as described when using the mask with ResMed VPAP Adapt SV.

N/V = The mask has not been validated for this device.

✓ = Compatible X = Not Compatible

Nasal Masks 1

Device	AirFit™ N20 / AirFit™ N20 for Her	AirFit™ N10 / AirFit™ N10 for Her	Swift™ FX Nano / Swift™ FX Nano for Her	Mirage™ FX / Mirage™ FX for Her	Mirage™ SoftGel	Mirage Activa™ LT	Mirage Micro™	Hospital Nasal	Pixi™
AirMini™	✓	X	X	X	X	X	X	X	X
Air 10 Series	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal	Nasal	N/V
S9 AutoSet™ / S9 AutoSet™ for Her / S9 Elite™	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal	Nasal	N/V
S9 Escape™ / S9 Escape™ Auto	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal	Nasal	N/V
C-Series Tango™	N/V	N/V	N/V	No setting required. Consult your clinician before changing to a different mask type.					N/V
S7 Elite™	N/V	N/V	N/V	STANDARD	STANDARD	STANDARD	STANDARD	STANDARD	N/V
S8 AutoSet Vantage™	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD	STANDARD	N/V
S8 Compact™	N/V	No setting required. Consult your clinician before changing to a different mask type.					N/V		
S8 Elite™ / S8 Elite™ II	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD	STANDARD	N/V
S8 Escape™	N/V	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.					N/V		
S8 Escape™ II	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.					N/V			
S8 AutoSet™ II	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD	STANDARD	N/V
VPAP™ III	N/V	MIRAGE	MIRAGE	STANDARD	STANDARD	STANDARD	STANDARD	N/V	N/V
VPAP™ III ST-A	N/V	MIRAGE	MIRAGE	STANDARD	STANDARD	STANDARD	STANDARD	N/V	N/V
VPAP Adapt SV™	N/V	N/V	N/V	VISTA ¹	ACTIVA	ACTIVA	VISTA ¹	VISTA	N/V
VPAP™ Adapt (S9)	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal	Nasal	N/V
VPAP™ Adapt (S9 ASV)	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal	Nasal	N/V
VPAP™ Auto 25	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD	STANDARD	N/V
VPAP™ Auto (S9)	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal	Nasal	N/V
VPAP™ Malibu	N/V	SWIFT	SWIFT	VISTA ¹	N/V	N/V	MERIDIAN	MERIDIAN	N/V
VPAP™ S / VPAP™ ST	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD	STANDARD	N/V
VPAP™ S (S9) / VPAP™ ST (S9)	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal	Nasal	N/V
VPAP™ ST-A (S9 iVAPS)	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal	Nasal	Pediatric
VPAP™ COPD (S9)	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal	Nasal	Pediatric
S9 VPAP™ Tx	N/V	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal	Nasal	Pediatric
VPAP™ Tx	N/V	PILLOWS	PILLOWS	NASAL	NASAL	NASAL	NASAL	NASAL	N/V
Lumis™ Tx	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal	Nasal	Pediatric

Device	AirFit™ N20 / AirFit™ N20 for Her	AirFit™ N10 / AirFit™ N10 for Her	Swift™ FX Nano / Swift™ FX Nano for Her	Mirage™ FX / Mirage™ FX for Her	Mirage™ SoftGel	Mirage Activa™ LT	Mirage Micro™	Hospital Nasal	Pixi™
Stellar™	PILLOWS	PILLOWS¹	PILLOWS¹	NASAL	NASAL	NASAL	NASAL	NASAL	Pediatric
VS Ultra™ / VS Integra™	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V
VS III™	N/V	N/V	N/V	N/V	N/V	✓	✓	✓	N/V
Astral™	Pillows	N/V	N/V	Nasal	Nasal	Nasal	Nasal	Nasal	Pediatric

¹ SmartStop may not operate as described when using the mask with ResMed VPAP Adapt SV / Stellar.

N/V = The mask has not been validated for this device. ✓ = Compatible X = Not Compatible

Nasal Masks 2

Device	Ultra Mirage™ II	Mirage Activa™	Mirage Vista™	Meridian	Mirage Kidsta™	Mirage Micro™ for Kids
AirMini™	X	X	X	X	X	X
Air 10 Series	Nasal	Nasal	Nasal	Nasal	Nasal	N/V
S9 AutoSet™ / S9 AutoSet™ for Her / S9 Elite™	Nasal Ultra	Nasal	Nasal	Nasal	N/V	N/V
S9 Escape™ / S9 Escape™ Auto	Nasal Ultra	Nasal	Nasal	Nasal	N/V	N/V
C-Series Tango™	No setting required. Consult your clinician before changing to a different mask type.			N/V	N/V	N/V
S7 Elite™	ULTRA	STANDARD	STANDARD	N/V	N/V	N/V
S8 AutoSet Vantage™	ULTRA	ACTIVA	STANDARD	STANDARD	N/V	N/V
S8 Compact™	No setting required. Consult your clinician before changing to a different mask type.				N/V	N/V
S8 Elite™ / S8 Elite™ II	ULTRA	ACTIVA	STANDARD	STANDARD	N/V	N/V
S8 Escape™	No setting required. Consult your clinician before changing to a different mask type.				N/V	N/V
S8 Escape™ II	No setting required. Consult your clinician before changing to a different mask type.				N/V	N/V
S8 AutoSet™ II	ULTRA	ACTIVA	STANDARD	STANDARD	N/V	N/V
VPAP™ III	ULTRA	STANDARD	STANDARD	N/V	N/V	N/V
VPAP™ III ST-A	ULTRA	STANDARD	STANDARD	N/V	STANDARD	STANDARD
VPAP Adapt SV™	ULTRA	ACTIVA	VISTA	VISTA	N/V	N/V
VPAP™ Adapt (S9)	Nasal Ultra	Nasal	Nasal	Nasal	N/V	N/V
VPAP™ Adapt (S9 ASV)	Nasal Ultra	Nasal	Nasal	Nasal	N/V	N/V
VPAP™ Auto 25	ULTRA	ACTIVA	STANDARD	STANDARD	N/V	N/V
VPAP™ Auto (S9)	Nasal Ultra	Nasal	Nasal	Nasal	N/V	N/V
VPAP™ Malibu	ULTRA	N/V	VISTA	MERIDIAN	N/V	N/V
VPAP™ S / VPAP™ ST	ULTRA	ACTIVA	STANDARD	STANDARD	N/V	N/V
VPAP™ S (S9) / VPAP™ ST (S9)	Nasal Ultra	Nasal	Nasal	Nasal	N/V	N/V
VPAP™ ST-A (S9 iVAPS)	Nasal Ultra	Nasal	Nasal	Nasal	N/V	N/V
VPAP™ COPD (S9)	Nasal Ultra	Nasal	Nasal	Nasal	N/V	N/V
S9 VPAP™ Tx	Nasal Ultra	Nasal	Nasal	Nasal	N/V	N/V
VPAP™ Tx	NASAL-ULTRA	NASAL	NASAL	N/V	NASAL	N/V
Lumis™ Tx	Nasal Ultra	Nasal	Nasal	Nasal	N/V	N/V
Stellar™	ULTRA	NASAL	NASAL	N/V	NASAL	N/V
VS Ultra™ / VS Integra™	N/V	N/V	N/V	N/V	N/V	N/V
VS III™	✓	✓	✓	N/V	✓	N/V
Astral™	Nasal Ultra	N/V	Nasal	N/V	Nasal	Nasal

N/V = The mask has not been validated for this device. ✓ = Compatible X = Not Compatible

Nasal Pillows

Device	AirFit™ P10 for AirMini™	AirFit™ P10 / AirFit™ P10 for Her	Swift™ FX / Swift™ FX for Her	Swift™ FX Bella / Swift™ FX Bella Gray	Swift™ LT / Swift™ LT for Her	Mirage Swift™ II
AirMini™	✓	X	X	X	X	X
Air 10 Series	X	Pillows ²	Pillows	Pillows	Pillows	Pillows
S9 AutoSet™ / S9 AutoSet™ for Her / S9 Elite™	X	Pillows	Pillows	Pillows	Pillows	Pillows
S9 Escape™ / S9 Escape™ Auto	X	Pillows	Pillows	Pillows	Pillows	Pillows
C-Series Tango™	X	No setting required. Consult your clinician before changing to a different mask type.				
S7 Elite	X	MIRAGE	MIRAGE	MIRAGE	MIRAGE	MIRAGE
S8 AutoSet Vantage™	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
S8 Compact™	X	No setting required. Consult your clinician before changing to a different mask type.				
S8 Elite™ / S8 Elite™ II	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
S8 Escape™	X	No setting required. Consult your clinician before changing to a different mask type.				
S8 Escape™ II	X	No setting required. Consult your clinician before changing to a different mask type.				
S8 AutoSet™ II	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ III	X	MIRAGE	MIRAGE	MIRAGE	MIRAGE	MIRAGE
VPAP™ III ST-A	X	MIRAGE	MIRAGE	MIRAGE	MIRAGE	MIRAGE
VPAP Adapt SV™	X	N/V	N/V	N/V	N/V	N/V
VPAP™ Adapt (S9)	X	Pillows	Pillows	Pillows	Pillows	Pillows
VPAP™ Adapt (S9 ASV)	X	Pillows	Pillows	Pillows	Pillows	Pillows
VPAP™ Auto 25	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ Auto (S9)	X	Pillows	Pillows	Pillows	Pillows	Pillows
VPAP™ Malibu	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ S / VPAP™ ST	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ S (S9) / VPAP™ ST (S9)	X	Pillows	Pillows	Pillows	Pillows	Pillows
VPAP™ ST-A (S9 iVAPS)	X	Pillows	Pillows	Pillows	Pillows	Pillows
VPAP™ COPD (S9)	X	Pillows	Pillows	Pillows	Pillows	Pillows
S9 VPAP™ Tx	X	Pillows	Pillows	Pillows	Pillows	Pillows
VPAP™ Tx	X	PILLOWS	PILLOWS	PILLOWS	PILLOWS	PILLOWS
Lumis™ Tx	X	Pillows	Pillows	Pillows	Pillows	Pillows
Stellar™	X	PILLOWS ¹	PILLOWS ¹	PILLOWS ¹	PILLOWS ¹	PILLOWS ¹
VS Ultra™ / VS Integra™	X	N/V	N/V	N/V	N/V	N/V
VS III™	X	N/V	N/V	N/V	N/V	N/V
Astral™	X	N/V	Pillows	Pillows	Pillows	Pillows

¹ SmartStop may not operate as described when using the mask with Stellar.

² Accuracy of tidal volume/minute ventilation may reduce using the AirFit P10 mask in bilevel modes (S, T, ST), particularly at higher pressure supports.

N/V = The mask has not been validated for this device.

✓ = Compatible X = Not Compatible



MANUFACTURER

Stellar **ResMed Germany Inc.** Fraunhoferstr. 16
82152 Martinsried Germany

VS III / VS Ultra /
VS Integra **ResMed Paris** 240 rue de la Motte 77550
Moissy-Cramayel France

All others **ResMed Ltd** 1 Elizabeth Macarthur Drive
Bella Vista NSW 2153 Australia

Distributed by

ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA
ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia

ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia

ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA
EC|REP ResMed (UK) (for Ltd) 96 Jubilee Ave Milton Park Abingdon
Oxfordshire OX14 4RW

See www.resmed.com for other ResMed locations worldwide. For patent and other intellectual property information, see www.resmed.com/ip. AcuCare, AirFit, S9, S9 AutoSet, S9 Elite, S9 Escape, AutoSet, Lumis Tx, Tango, S8, S8 AutoSet Vantage, S8 Escape, S8 Elite, S8 Compact, VPAP, VPAP Adapt SV, VPAP Tx, Stellar, Mirage, Mirage Quattro, Mirage Liberty, Mirage Micro, Mirage Vista, Mirage Kidsta, Ultra Mirage, Activa, Pixi, Swift, Mirage Swift and Quattro are trademarks of ResMed Ltd and Astral, Tango, AutoSet, Escape, S9, VPAP, Adapt SV, Stellar, Mirage, Mirage Quattro, Quattro, Activa, Kidsta, Pixi, Swift and Vista are registered in U.S. Patent and Trademark Office. Astral, VS Ultra, VS Integra and VS III are trademarks of ResMed Paris SAS.
© 2017 ResMed Ltd. 1017959/11 2017-05